

Lens-Grip

Liner
83.954

Directions

1. Make sure the lens is clean. If necessary, clean with Acetone or Isopropyl Alcohol
 2. Apply Lens-Grip around the edges of lens
 3. Cut Lens-Grip at desired length
 4. Press the white paper firmly against the edge of the lens
 5. Peel off the white paper from the lens (the thin adhesive strip remaining on the lens edge is the light weight Lens-Grip material)
 6. If extra thickness is desired, repeat steps 2-5
 7. Place lens in frame and secure all screws.
-

VIGOR[®]

www.vigoroptycal.com

Carlstadt, NJ 07072